

Synapse Mobility

How to use Synapse Mobility to view studies:




What is Synapse Mobility (SynMobility)?

Synapse Mobility enables physicians and clinical users to view the same high-quality, interactive images that can be seen at a diagnostic workstation, but in a convenient and readily accessible form factor.




Accessing & Using Synapse Mobility via EPIC


1. On the imaging tab in Chart Review, Click the Synapse Imaging button to open the image viewer.

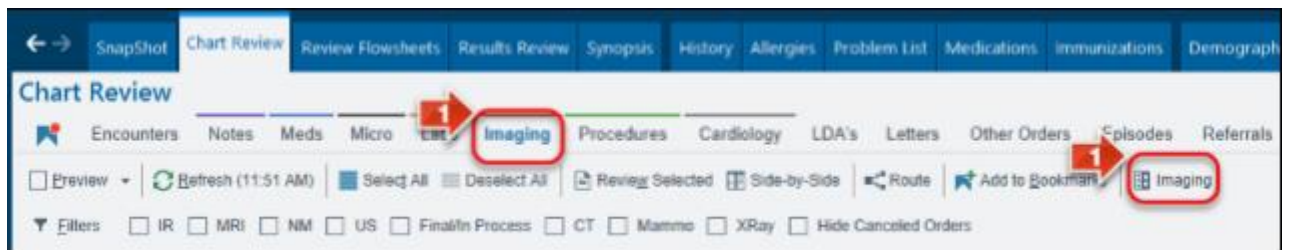


2. The image viewer opens with all the studies available.
 - a. Hover over the edge to expand your screen to view images better and build your worklist.

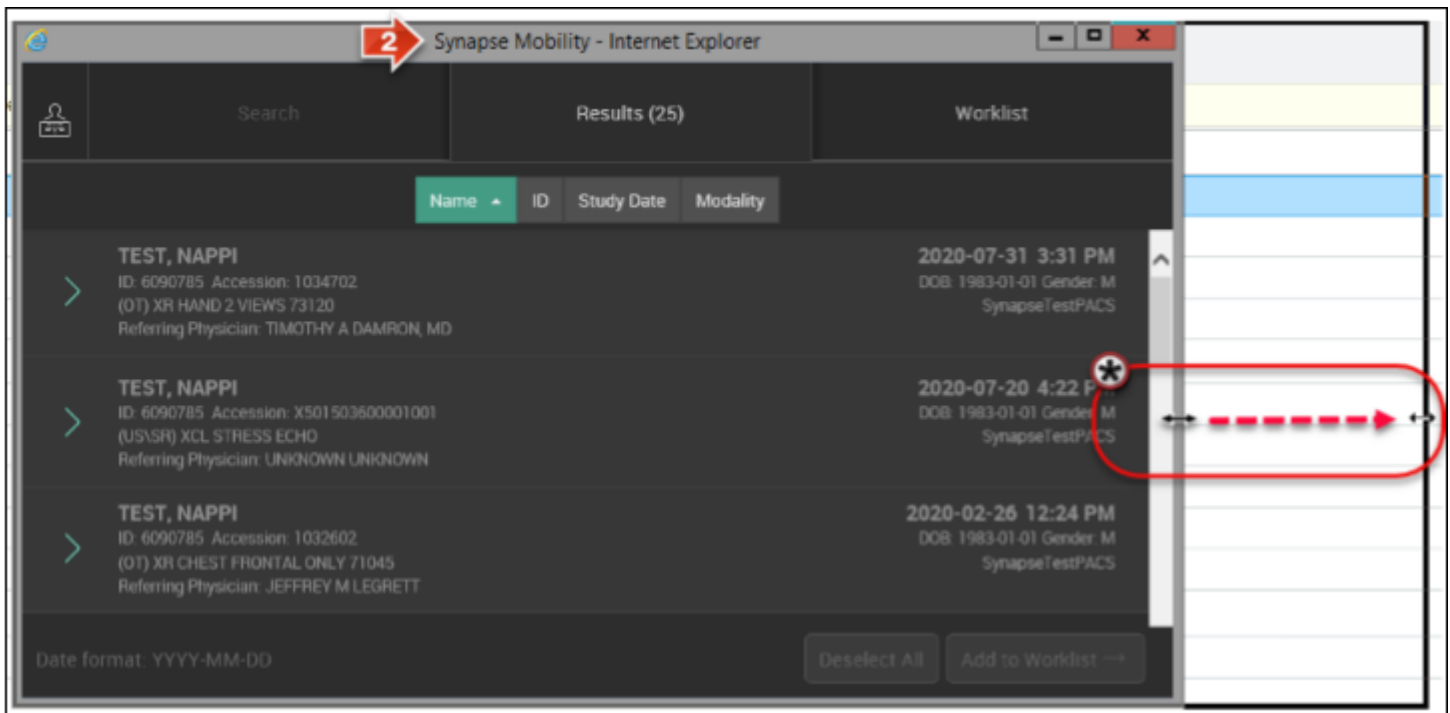

3. Click the arrow to expand the study view.
4. When the study opens, select the images you would like to display
5. Click Add to worklist.

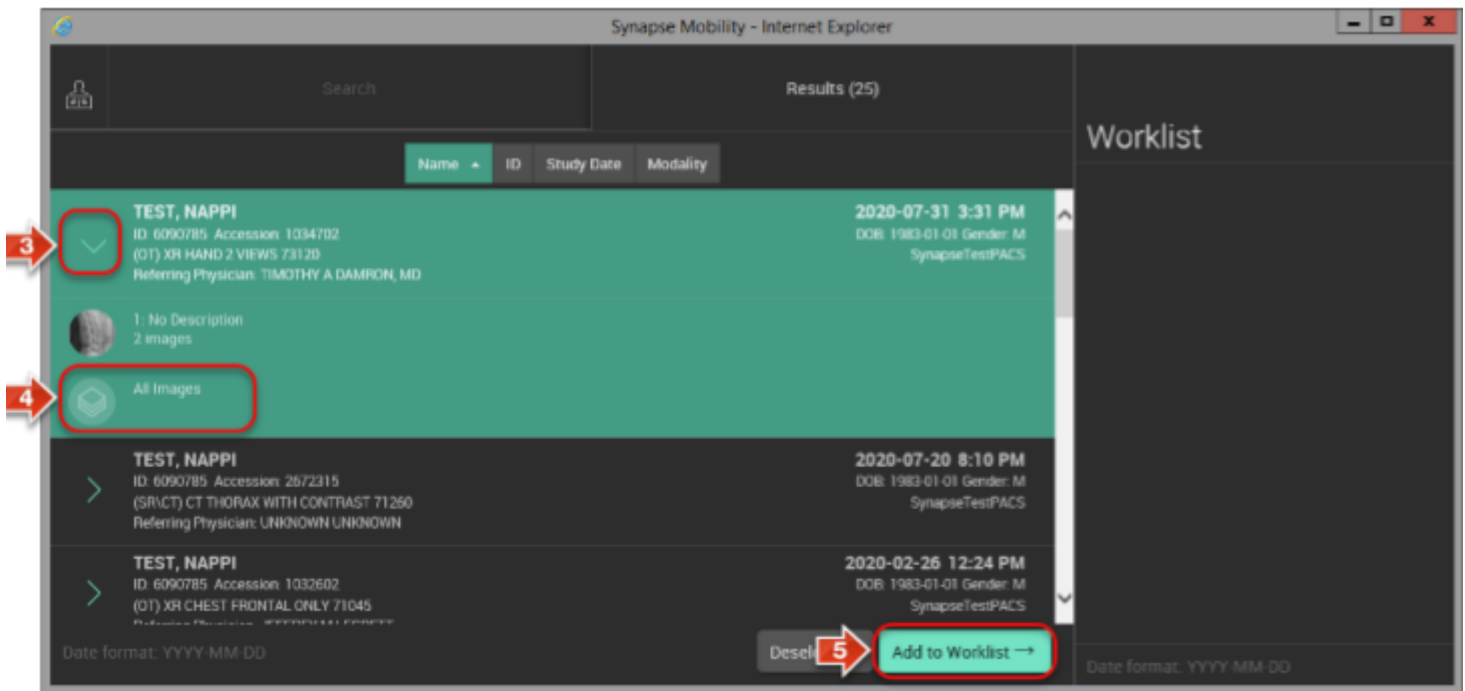

6. From the worklist, expand the study and click the images
7. The images will display and you can choose how many you want to view at one time.
8. Use the Icons at the bottom to choose the display you want.

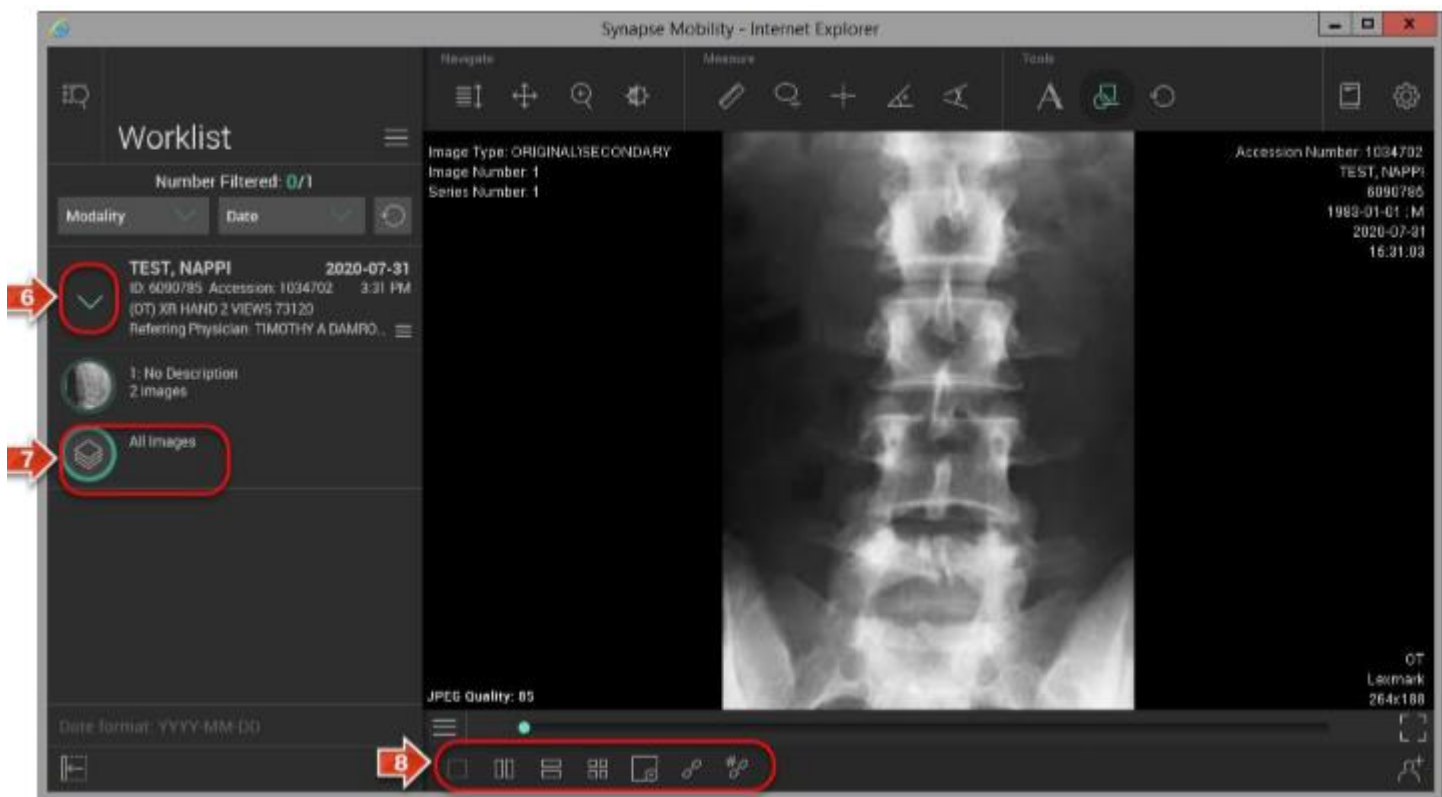

9. Drag Images to the viewing port.

