

SynMobility Viewer


Locate the image(s) in Chart Review

1. Locate the patient's chart review in EPIC

The screenshot displays the EPIC SynMobility Viewer interface. The top navigation bar includes various tabs like Record Viewer, Patient Station, Tech Work List, Study Review, Study History, Anc Orders, Recent Exams, Recent Protocols, Workqueues, Status Report, Unsigned Studies, Transcriptionist Work List, Snapboard, Front Desk, Schedule, and Log Out. The patient information header shows details for Zztstepic, Production "Techo", including gender, date of birth, MRN, and vital signs. The main content area is titled "Chart Review" and includes a sidebar with navigation options like Encounters, Notes, Letters, Meds, Labs, Micro, Imaging, Radiology, Procedures, LDA's, Other Orders, Episodes, Referrals, Media, Adv Directives, Misc Reports, and Pat Sum Extracts. The "Imaging" button is highlighted with a red box and the number 2. The "Chart Review" button is highlighted with a red box and the number 1. The table of exams lists various studies, with the entry "US GUIDED THORACENTESIS (IN..." highlighted by a red box and the number 3. The "PACS Images" link is highlighted with a red box and the number 4.

Critical	Order Date	Accession#	Performing Location	Exam	Description
	10/20/2018			XR CHEST FRONTAL AND LATERA...	
	10/09/2018			CT SIMULATION AT RAD ONC (IN O...	
	10/02/2018			DEXA HIP AND OR SPINE 77080	
	10/02/2018			DEXA HIP AND OR SPINE 77080	
	10/01/2018	THOU113733123		US GUIDED THORACENTESIS (IN...	
	09/26/2018			US GUIDED THORACENTESIS (IN...	
	09/26/2018	THOU96487614		US GUIDED THORACENTESIS (IN...	
	09/26/2018			US GUIDED THORACENTESIS (IN...	
	09/26/2018			US GUIDED THORACENTESIS (IN...	
	09/26/2018			US GUIDED THORACENTESIS (IN...	
	07/30/2018			ARTHROCENTESIS OF MAJOR JOI...	
	07/30/2018			ARTHROCENTESIS OF MAJOR JOI...	
	07/12/2018	RHU96487608		ARTHROCENTESIS OF INTERMEDI...	
	07/12/2018			ARTHROCENTESIS OF INTERMEDI...	
	07/12/2018			ARTHROCENTESIS OF SMALL JOI...	
	07/11/2018	MLD96487602		US GUIDED THORACENTESIS AT...	
	07/11/2018	MLD96487603		US GUIDED THORACENTESIS AT...	
	07/11/2018			US GUIDED THORACENTESIS AT...	
	07/03/2018			US GUIDED THORACENTESIS AT P...	
	07/03/2018			ARTHROCENTESIS OF MAJOR JOI...	
	07/03/2018			ARTHROCENTESIS OF INTERMEDI...	

2. Click the "Imaging" button
3. Locate the patient's exam
4. Locate the hyperlink referencing the images and left-click it

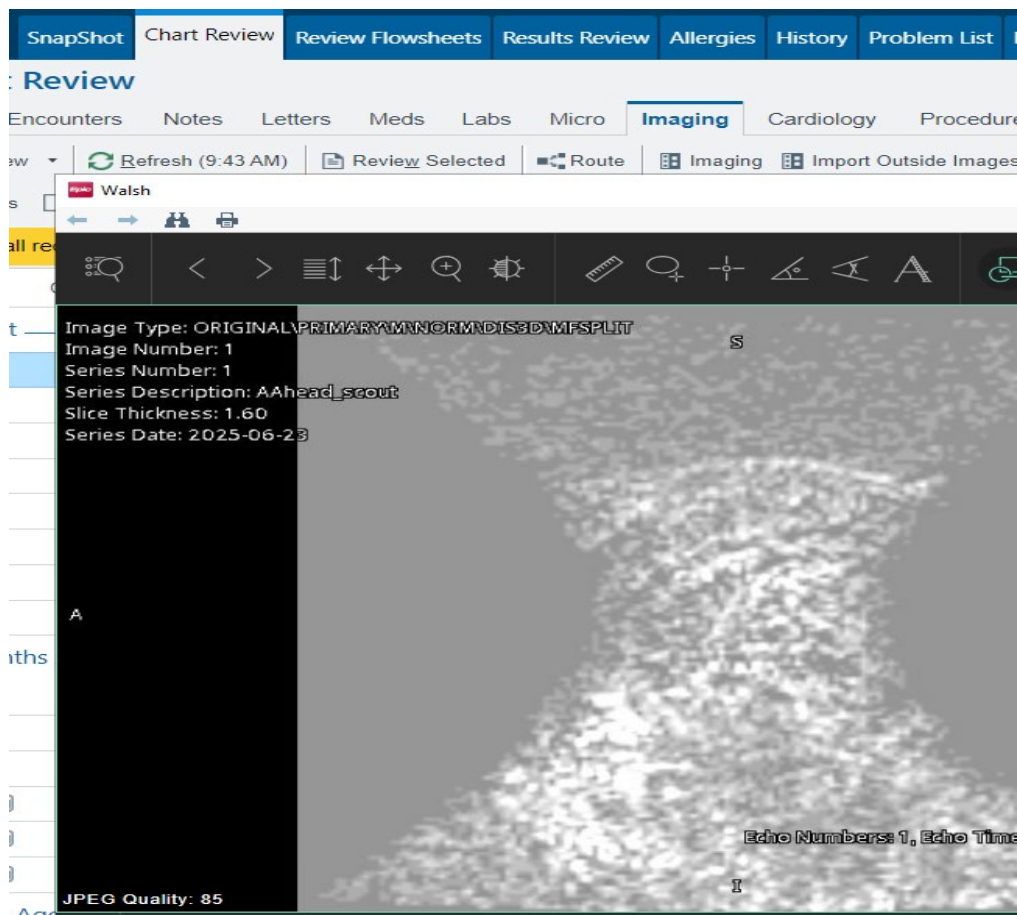

Accessing image(s) using Synapse Mobility

